



MICROBIOLOGY SPECIMEN COLLECTION

YELLOW SWAB:

RAPID STREP SCREEN/THROAT CULTURE
FEMALE GENITAL CULTURE/WET PREP
AEROBIC CULTURE → **DOCUMENT SITE**



BLUE SWAB:

AEROBIC/ANAEROBIC CULTURE
→ **DOCUMENT SITE**



REMEL WHITE SWAB:

HERPES CULTURE



STERILE CONTAINER:

URINE CULTURE
SPUTUM CULTURE
AFB CULTURE
BIOPSY FOR CULTURE-DOCUMENT BIOSPY SITE
FRESH STOOL SAMPLE



BD TUBE: FLUID FOR CULTURE- DOCUMENT FLUID SITE

UTM MEDIA:

SARS-CoV-2, INFLUENZA A/B, and RSV by PCR

NASAL COLLECTION

→ SPECIMEN MUST BE REFRIGERATED
→ CALL LAB TO OBTAIN COLLECTION KIT



COPAN RED SWAB:

MRSA SURVEILLANCE by PCR

Nasal collection ONLY




APTIMA SWAB:

STD → GC/CHLAMIDIA

URINE=YELLOW OR STERILE CONTAINER

CERVICAL/VAGINAL = PINK

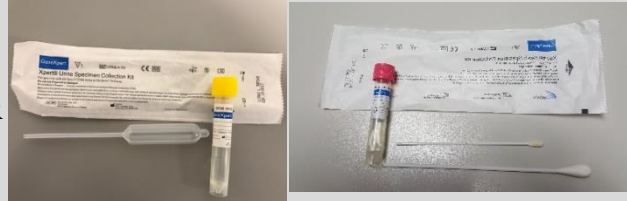

ORANGE PARA PAK:

STOOL CULTURE

C DIFFICILE

OVA & PARASITE (NO TRAVEL HISTORY)

*NOTE: FILL CONTAINER ONLY TO RED LINE



PARA PAK FORMALIN AND PVA:

OVA & PARASITE (KNOWN TRAVEL HISTORY)

- DOCUMENT LOCATION

- MUST COLLECT BOTH VIALS

*NOTE: FILL CONTAINER ONLY TO RED LINE



UTM MEDIA:

BORDETELLA by PCR

NASOPHARYNGEAL COLLECTION

→ SPECIMEN MUST BE REFRIGERATED

→ CALL LAB TO OBTAIN COLLECTION KIT

